

POWERED AND ILLUMINATED FOOTREST



The new powered footrest upgrades both the comfort and safety of the Handicare Elegance and Alliance seat. It makes it much easier to sit down or stand up and allows you to raise and lower the footrest automatically whilst seated, using the switch in the seat arm. Moreover, the unique LED lighting illuminates the footrest at night, making it safer for other users of the stairs.

How does the powered footrest work?

The powered footrest option offers you easier access to the stairlift. With the footrest folded, you can stand close to the seat, without having to position your feet up or around the footrest before sitting down. There is no need to bend down to unfold or fold the footrest since you can simply use the armrest switch to operate the powered mechanism when sitting on the seat or standing next to the stairlift.

When not in use, a folded footrest maximizes the clear space on the stairs or in the hallway making it easier for other users of the stairs to pass the stairlift.

Stylishly illuminated footrest for your safety

The unique lighting feature illuminates the top surface of the unfolded footrest, improving the visibility in the dark. The LED light switches on automatically when the footrest is unfolded. This safety feature prevents you or other stair users from stumbling over the unfolded footrest in the dark.

Easy to use

For easy access the footrest switch is located in the left or right hand armrest, adjacent to the joystick. The powered option is available for all standard and offset footrests. These are fitted with safety sensors that will directly stop the movement of the footrest when it encounters an obstruction. The movement will continue when the obstacle, e.g. your shoe, is out of the way. Although the powered footrest has an automated folding mechanism for your convenience, the footrest can still be operated manually by simply lifting it up or pushing it down when required.



The described product will be introduced in 2015. Please contact your Handicare account manager for more information about the availability in your country. All information is provided in good faith but does not constitute a guarantee. Handicare reserves the right to alter designs and specifications without prior notice.



Bentley Mobility Services Ltd



Unit 19 Highview Business Centre
High Street
Bordon
Hampshire GU35 0AX

T: 01420 470730
F: 01420 470731
W: www.bentleymobility.co.uk