

# Saxon



Straight stairlift

# Stannah

**My stairs are quite narrow,  
but the Saxon folds away neatly,  
leaving plenty of room for the  
grandchildren to get up and down.**



## Make a step change

### If a Stannah stairlift could make life easier, why wait?

Life is better when you have the energy to do the things you want to do, like enjoying a visit from the grandchildren or entertaining friends. Our experience has taught us that the sooner a stairlift is installed, the sooner you will be able to start enjoying the things you love doing. Just think how a Stannah stairlift could make your day-to-day life easier.

#### A stairlift that's right for you and your home.

We know that not all homes are the same. If you think your stairs are unsuitable for a stairlift, we think you might be pleasantly surprised.

- Our slimline rails and fold-away chairs make the best use of available space, leaving enough room for others to use the staircase.
- Contrary to popular belief our rails are fitted to the staircase and not the wall – ensuring minimal mess and bother.
- Installation is quick and easy.

**“I wonder  
how I ever  
managed  
without it.”**

**Mrs Clarke, 2011**





**Over 500,000  
stairlifts sold  
globally**



## **A company you can trust**

We are a British, family-run company and have been for five generations. The current managing directors are the great, great grandchildren of Joseph Stannah, who started the company in the 1860s. His engineering principles and strong family values run as true today as they always have.

Now, 140 years later, we have manufactured more than 500,000 stairlifts and installed products in over 40 countries. The half-million milestone was commemorated with a visit from HRH The Prince of Wales – a proud day for all those who work at Stannah.

## **The Saxon**

The Saxon is one of our most popular stairlifts. It is perfect for straight staircases, even narrow ones, whilst being comfortable and easy-to-use.

Its design means that it won't obstruct your hallway or stairs, and you can pick from a choice of upholstery colours so it suits the interior of your home.

## Key features

### 1. Arm-to-footrest link

When not in use, your Saxon stairlift can be folded away by folding the arm. The footrest will automatically fold with it. There's no awkward bending down, which is particularly useful when at the top of the stairs.

### 2. Adjustable seat height

The adjustable seat height can be set to the most comfortable setting for you when the stairlift is installed, allowing you to get on and off the chair easily.

### 3. Safety edges

In the event that something obstructs the stairlift, sensors on the edge of the carriage and footrest will automatically bring the stairlift to a safe stop.

### 4. Retractable seatbelt

Fasten the seatbelt for added security.



### 5. Directional controls

Right-handed or left-handed, we can put the controls on either arm to suit you.



### 6. Wall controls

Ideal for when you need to summon your stairlift from another floor, or perhaps send it up or down to someone else who wants to use it.

**“It has been so refreshing to deal with a firm that appreciates what I call old fashioned values, superb service with a first class product.”**

**Mr Tyson, 2011**



## Using your Saxon

The simple-to-use Saxon stairlift has been designed to make your life easier and will help you to rediscover your independence. This step-by-step guide demonstrates some of its key features.



1. The wall-mounted controls at the top and bottom of the stairs make it easy to call the stairlift wherever you are.



2. Simply unfold the seat, followed by the arms. The arm-to-footrest link will automatically lower the footrest down too.



3. Once unfolded, make yourself comfortable in the seat. Fasten your seatbelt for extra security.



4. Gently apply pressure to the control in the direction you wish to travel and your Saxon will glide smoothly up or down the stairs.



5. When at the top of the stairs, the footrest will stop level with your landing. You can then swivel the chair to face the landing to alight safely.




6. For added safety, the Saxon can remain in the swivelled position to form a barrier at the top of the stairs.



7. When not in use, your Saxon folds neatly against the wall, giving you and others plenty of space.

# Your Saxon - tailored for you

## Standard Package

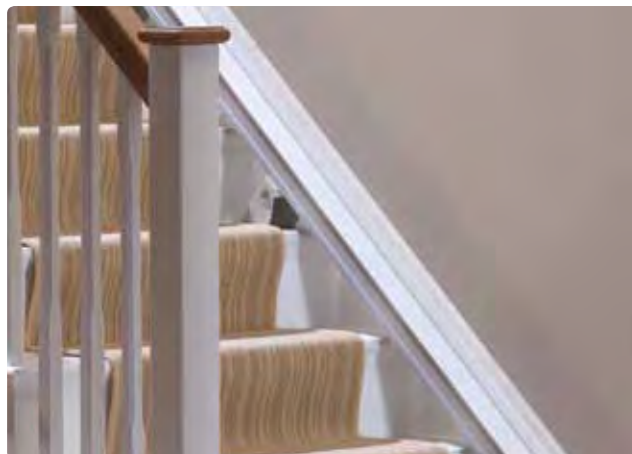

Straight silver rail

**In addition to the standard package, you can choose from a range of other optional extras that will ensure your Saxon is just right for you. Our sales consultant will guide you through the various options available to you.**



Retractable seatbelt



Manual swivel seat



Directional controls – joystick




Arm-to-footrest link


Left hand  Right hand

# Upholstery colours and optional extras


## Vinyl upholstery

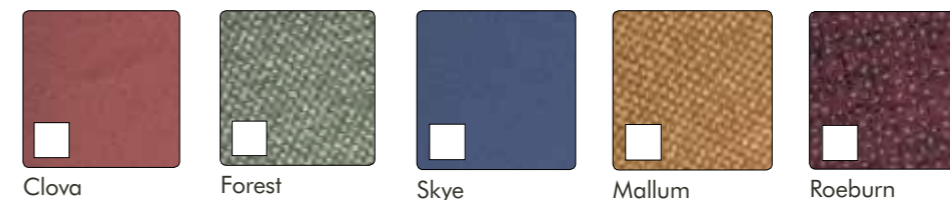


## Straight rail colours

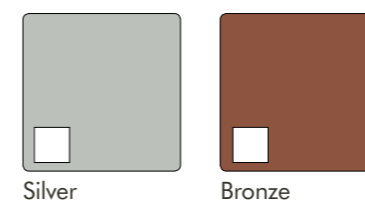

## Rail start

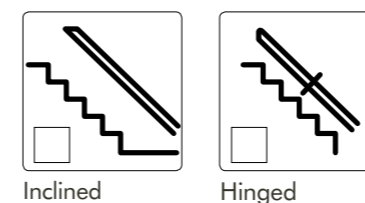

## Rail finish

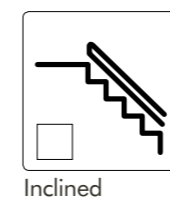

## Swivel



Powered

## Controls



Push button

## Seatbelt




Diagonal lap harness



Five-point harness

## Seat

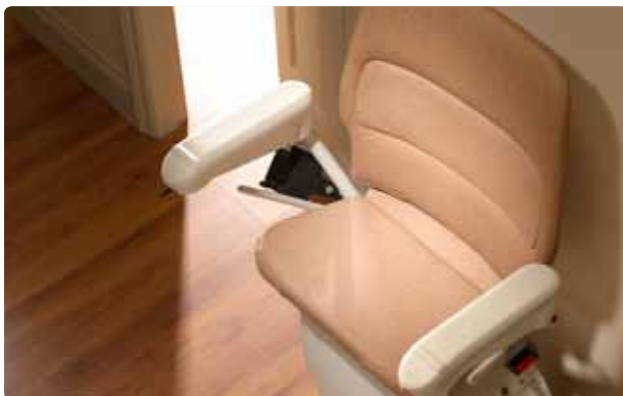

Reduced seat  
(heightened seat option also available)

## Dimensions

Min. width of unfolded stairlift:  
644.5mm / 25.4"

Min. width of folded stairlift:  
330mm / 13"

Max. weight range:  
120 – 160kg / 19 – 25 stone



Bentley Mobility Services Ltd



Unit 19 Highview Business Centre  
High Street  
Bordon  
Hampshire GU35 0AX

T: 01420 470730  
F: 01420 470731  
W: [www.bentleymobility.co.uk](http://www.bentleymobility.co.uk)

